




J DENTAL CARE

*just smile*



MADE IN ITALY

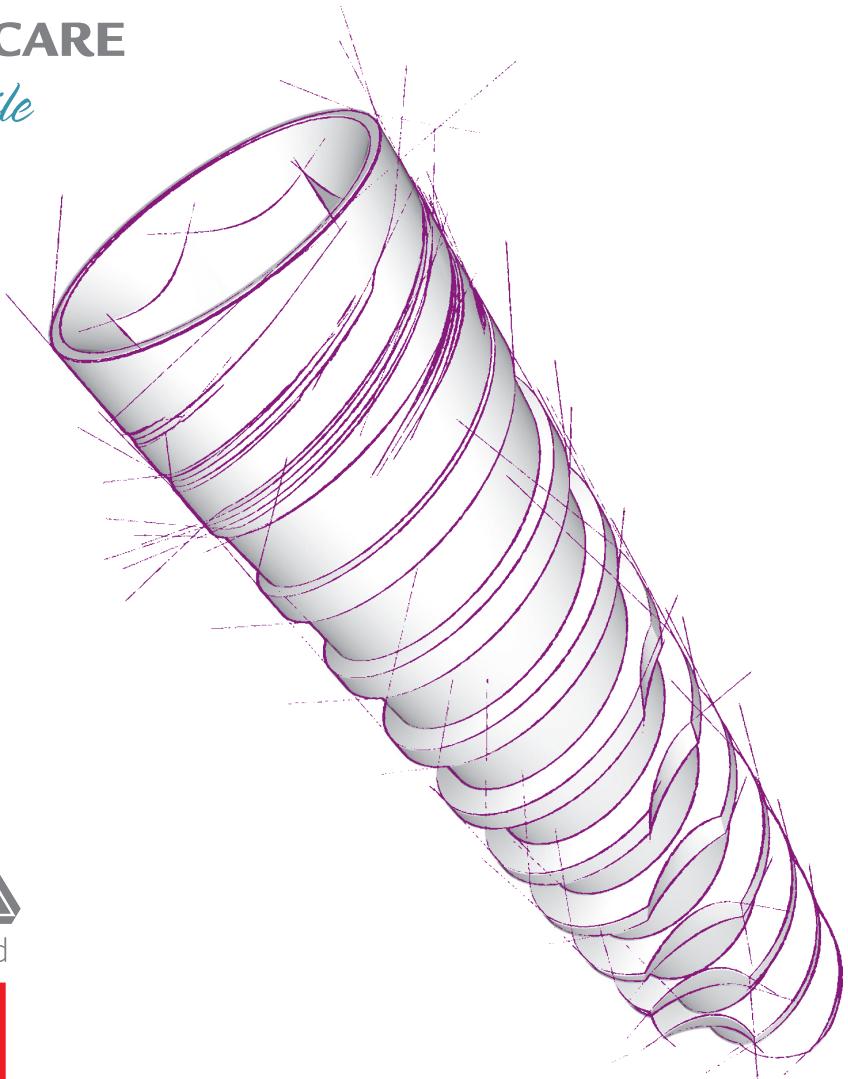


Dental Implant

**JD EVOLUTION<sup>®</sup>S**

# JD EVOLUTION® S

## SAFE AS ALWAYS, THINNER THAN EVER

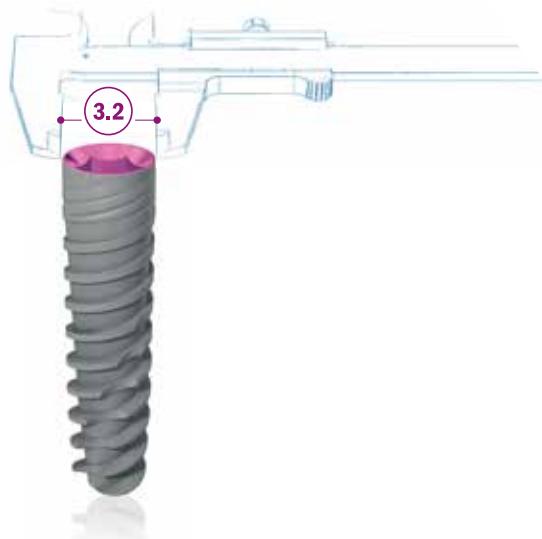


### ADVANTAGES

- High primary stability, even in type 4 bone.
- Self drilling and self cutting.
- Redirecting capability for optimal placement.
- Immediate function capability.
- Built-in Platform switching.
- Solves tight spacing problems.
- Designed specifically for narrow spaces in the esthetic zone.

### IMPLANT MEASURES AND LENGTHS

The JDEvolution® S dental implant is available in the lengths shown in the following chart:

$\varnothing 3.2$ mm	8 mm		10 mm		11,5 mm		13 mm		15 mm
----------------------	------	--	-------	--	---------	--	-------	--	-------

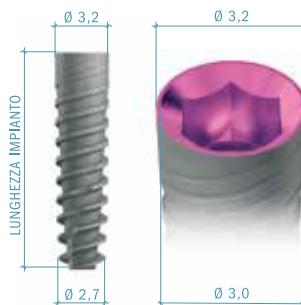

### DRILLING SEQUENCE

It is recommended to adhere to the indications of the following drilling sequence to ensure optimal primary stability of the implant.

IMPLANT DIAMETER	SOFT BONE TYPE IV	MEDIUM BONE TYPE II-III	DENSE BONE TYPE I
$\varnothing 3.2$	1,5 (2)	2 2,4	2 2,4 (2,8)

Note: All dimensions are expressed in millimeters.

# PRODUCT CATALOGUE

## Implants:

**Ø 3,2**

- |          |                             |
|----------|-----------------------------|
| EV32 080 | JDEvolution® S Ø 3.2 L 8    |
| EV32100  | JDEvolution® S Ø 3.2 L 10   |
| EV32115  | JDEvolution® S Ø 3.2 L 11,5 |
| EV32130  | JDEvolution® S Ø 3.2 L 13   |
| EV32150  | JDEvolution® S Ø 3.2 L 15   |



## Cover screw:

- |      |                            |
|------|----------------------------|
| ESCS | Cover screw JDEvolution® S |
|------|----------------------------|



## Healing Abutment:


- |         |   |
|---------|---|
| ESHA323 | Healing Abutment Ø 3.2 H 3.0 JDEvolution® S |
| ESHA325 | Healing Abutment Ø 3.2 H 5.0 JDEvolution® S |
| ESHA327 | Healing Abutment Ø 3.2 H 7.0 JDEvolution® S |
| ESHA403 | Healing Abutment Ø 4.0 H 3.0 JDEvolution® S |
| ESHA405 | Healing Abutment Ø 4.0 H 5.0 JDEvolution® S |
| ESHA407 | Healing Abutment Ø 4.0 H 7.0 JDEvolution® S |



## Impression coping:

**OPEN TRAY**

- |           |  |
|-----------|--|
| ESICOT32C | Impression coping open tray Ø 3.2 JDEvolution® S |
| ESICOT40C | Impression coping open tray Ø 4.0 JDEvolution® S |

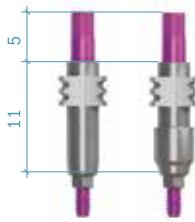

**CLOSED TRAY**

- |           |  |
|-----------|--|
| ESICCT32C | Impression coping closed tray Ø 3.2 JDEvolution® S |
| ESICCT40C | Impression coping closed tray Ø 4.0 JDEvolution® S |



## Temporary Abutment:

- |         |  |
|---------|--|
| ESTAEC  | Temporary Abutment engaging JDEvolution® S     |
| ESTANEC | Temporary Abutment non engaging JDEvolution® S |



## Immediate Temporary Abutment:

- |         |  |
|---------|--|
| ESIT15C | Immediate Temporary Abutment h1.5 JDEvolution® S |
| EVITC   | Healing cap                                      |



## GP Abutment:

ESGPAE32C	GP Abutment Ø 3.2 JDEvolution® S
ESGPAE40C	GP Abutment Ø 4.0 JDEvolution® S
ES15GPAE40C	GP Abutment 15° Ø 4.0 JDEvolution® S



## Anatomic Abutment:

ESAA15C	Anatomic Abutment Ø 4.0 H 1.5 JDEvolution® S
ESAA30C	Anatomic Abutment Ø 4.0 H 3.0 JDEvolution® S



## Wax up Abutment:

ESWAEC	Wax-up Abutment engaging JDEvolution® S
ESWANEC	Wax-up Abutment non engaging JDEvolution® S



## Prosthetic screw:

ESS	Abutment Screw JDEvolution®S
ESSA	Abutment Screw Angulated JDEvolution®S



## Conical Abutment:

ESCA15C	Straight Conical Abutment H 1.5 JDEvolution®S
ESCA1725C	17° Angulated Conical Abutment H2.5 JDEvolution®S
ESCA3025C	30° Angulated Conical Abutment H2.5 JDEvolution®S
ESCASA	Screw for angulated Conical Abutment JDEvolution®S



## TiBase, JDScanBody and Implant Replica CAD CAM:

ESSBCEC	JDScanBody for JDEvolution®S
ESTIBC	TiBase engaging JDEvolution®S
ESTIBNEC	TiBase non engaging JDEvolution®S
ESTIB15C	TiBase engaging H 1.5 JDEvolution®S
ESTIB15NEC	TiBase non engaging H 1.5 JDEvolution®S
ESTIB30C	TiBase engaging H 3.0 JDEvolution®S
ESTIB30NEC	TiBase non engaging H 3.0 JDEvolution®S
EVSCBOW	ScanBody for TiBase
ESANC	Implant replica CAD CAM for JDEvolution®S
ESPMNC	JD Pre-milled blanks and abutment screw JDEvolution®S



## Gold Abutment:

ESGAEC

Gold Abutment engaging JDEvolution® S



## Ball Abutment:

ESBA15 Ball Abutment H 1.5 JDEvolution® S

ESBA30 Ball Abutment H 3.0 JDEvolution® S

ESBA50 Ball Abutment H 5.0 JDEvolution® S

EVBAAB Ball Abutment replica JDEvolution® S

EVBAHC Cap attachment housing and elastic retentive cap Ball Abutment

EVBAN Elastic retentive cap Ball Abutment



## Emi Abutment:

ESEMI15 Emi Abutment H 1.5 JDEvolution® S

ESEMI30 Emi Abutment H 1.5 JDEvolution® S

ESEMI50 Emi Abutment H 1.5 JDEvolution® S

EVEMIA Emi Abutment replica JDEvolution® S

EVEMISCK Smart cap attachment housing Emi Abutment

EVEMIHC Cap attachment housing and elastic retentive cap Emi Abutment

EVEMIN Elastic retentive cap Emi Abutment



## Implant replica:

ESAN Implant replica JDEvolution® S



## Implant and Prosthetic Drivers

ESID Implant Driver JDEvolution® S

ESIDL Implant Driver Long JDEvolution® S

EVSDPF20 Screwdriver for torque wrench L20

EVSDPF25 Screwdriver for torque wrench L25

EVSDCAF Screwdriver torque wrench Conical Abutment

EVSDP20 Screwdriver machine prosthetic L20

EVSDP25 Screwdriver machine prosthetic L25

EVSDP30 Screwdriver machine prosthetic L30

EVSDCA Screwdriver machine Conical Abutment

EVSDPF25A Screwdriver screw angulated for torque wrench L25

EVSDPF30A Screwdriver screw angulated for torque wrench L30





**J DENTAL CARE**

[www.jdentalcare.com](http://www.jdentalcare.com)